
**LIGACLIP®**

**Endoscopic Clip Applier**

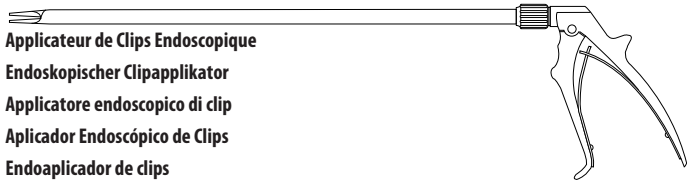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**Aplikátor endoskopických svorek**

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---

**Please read all information carefully.**

Failure to properly follow the instructions may lead to serious surgical consequences, such as failure to ligate.

**Important:** This package insert is designed to provide instructions for use of the LIGACLIP® Endoscopic Clip Applier. It is not a reference to ligation techniques.




LIGACLIP is a trademark of Ethicon Endo-Surgery.

**ETHICON ENDO-SURGERY, LLC**

a *Johnson & Johnson* company

**Instructions, Instructions, Gebrauchsanweisung, Istruzioni, Instruções, Instrucciones, Gebruiksaanwijzing, Brugsvejledning, Ohje, Οδηγίες, Bruksanvisning, Instrukcja, Utasítások, Návod k použití, Návod, 使用说明**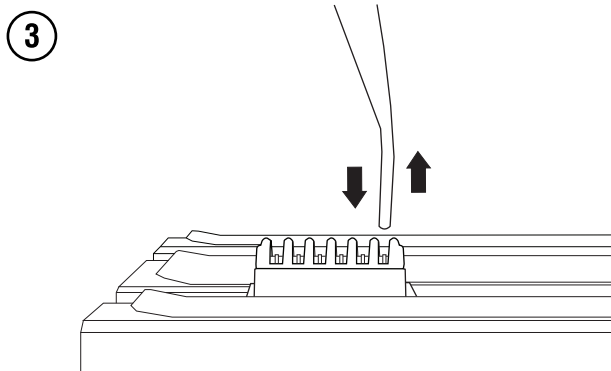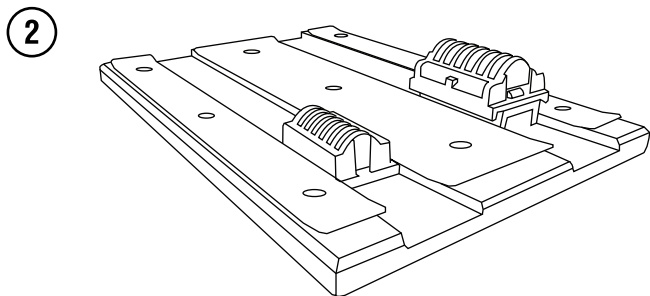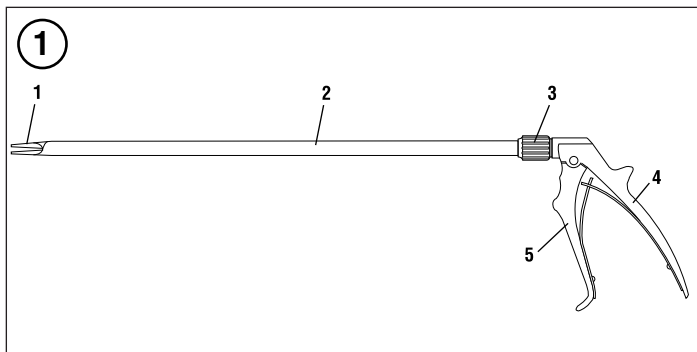

## Indications

The LIGACLIP Endoscopic Clip Applier has application for use on tubular structures or vessels wherever metal ligating clips are indicated. The tissue being ligated should be consistent with the size of the clip.

## Contraindications

- Do not use the instrument for contraceptive tubal occlusion.
- Do not use the instrument on tissue structures or vessels upon which metal ligating clips would not normally be used.

## Device Description

The LIGACLIP Endoscopic Clip Applier is a reusable instrument designed to deliver and close metallic ligating clips. This instrument is for use with LIGACLIP EXTRA Ligating Clips which are produced from titanium. The clips are designed for the ligation of tubular structures where a nonabsorbable ligating device is indicated.

The ligating clips are positioned around a tubular structure and closed by applying pressure to the trigger of the applier.

## Illustration and Nomenclature (Illustration 1)

- |    |               |    |         |
|----|---------------|----|---------|
| 1. | Jaws          | 4. | Handle  |
| 2. | Shaft         | 5. | Trigger |
| 3. | Rotation Knob |    |         |

## Instructions for Use

Verify compatibility of all instruments and accessories prior to using the instrument (refer to **Warnings and Precautions**).

- 1 Remove the protective cap from the instrument prior to sterilization.
- 2 Using sterile technique, remove the instrument from the sterile wrapping. To avoid damage, do not flip the instrument into the sterile field.
- 3 Select the appropriate size LIGACLIP EXTRA Ligating Clip cartridge and corresponding clip applier.
- 4 With the cartridge slots facing away from the base, insert the cartridge into one of the channel openings of the LC800 stainless steel cartridge base. Ensure that the cartridge is past the channel opening and securely held in place (Illustration 2).
- 5 Grasp the applier by the handle and trigger and insert the jaws of the instrument into the individual cartridge slot, making sure the tips of the applier are perpendicular to the surface of the cartridge (Illustration 3). Insert the applier until it stops. Do not force the applier. It should enter and withdraw from the cartridge smoothly.
- 6 Keeping the applier jaws perpendicular to the surface of the cartridge, retract the applier from the cartridge (Illustration 3). The clip will be securely held in the applier jaws. It is not necessary to maintain trigger tension to hold the clip in the applier jaws.
- 7 The sequence for rotation of the applier shaft is as follows:
  - Loosen the rotation knob by turning it clockwise until a click is heard.
  - Rotate the shaft to the desired position.
  - Tighten the rotation knob completely by turning it counterclockwise.
- 8 Position the clip around the tubular structure to be ligated. Apply sufficient force to fully close the applier to assure that the clip is satisfactorily placed and secured.

## Cleaning, Disinfection, and Sterilization

**The user must ensure that cleaning and sterilization are conducted in accordance with the appropriate guidelines, standards, or national Health Authorities' requirements.** The LIGACLIP Clip Applier must be cleaned and sterilized prior to each use. The instrument has been designed for thorough cleaning and safe sterilization.

Immediately after removing the instrument from the patient, immerse the entire instrument in tap or purified water. To ensure adequate cleaning, it is imperative that the instrument is not allowed to dry soiled.

All surgical instruments are subject to a degree of wear and tear as a result of normal use. A regular and precise visual check of the instrument should be made before each use.

#### **Manual Cleaning**

- 1 Rinse the instrument thoroughly with tap water.
- 2 Prepare a neutral pH enzymatic solution per manufacturer's recommendations. In addition, Neodisher® Mediclean Forte can be used. Purified or deionized water should be used throughout the cleaning process when using Neodisher® Mediclean Forte. Immerse the LIGACLIP Clip Applier in the solution. Actuate the instrument several times with the distal end pointed downward. Use a soft bristle brush or soft cloth to manually clean the device while it is immersed in the enzymatic solution, paying particular attention to crevices, seams, and joints. Leave the device in the solution for a minimum of 10 minutes.
- 3 Remove the instrument and rinse thoroughly under running tap water. Actuate the handle several times during rinsing.
- 4 Place the LIGACLIP Clip Applier in an ultrasonic cleaner with fresh enzymatic detergent for a minimum of 10 minutes. Ensure that the distal end of the instrument is pointed downward. Rinse thoroughly under running tap water while actuating the handle.
- 5 Prepare a neutral pH detergent (or Neodisher® Mediclean Forte) according to manufacturer's recommendations. Immerse the LIGACLIP Clip Applier in the cleaning solution. Actuate the instrument several times with the distal end pointed downward. Use a soft bristle brush or soft cloth to manually clean the device while it is immersed in the cleaning solution, paying particular attention to crevices, seams, and joints.
- 6 Rinse the instrument thoroughly under warm running tap water.
- 7 Dry the device with a clean absorbent cloth.
- 8 Prior to use, the instrument should be soaked in an instrument milk bath or similar solution to maintain proper lubrication of moving parts.

#### **Automated Cleaning**

- 1 Rinse the instrument thoroughly with tap water.
- 2 Prepare a neutral pH enzymatic solution per manufacturer's recommendations. In addition, Neodisher® Mediclean Forte can be used. Purified or deionized water should be used throughout the cleaning process when using Neodisher® Mediclean Forte. Immerse the LIGACLIP Clip Applier in the solution. Actuate the instrument several times with the distal end pointed downward. Use a soft bristle brush or soft cloth to manually clean the device while it is immersed in the enzymatic solution, paying particular attention to crevices, seams, and joints. Leave the device in the solution for a minimum of 10 minutes.
- 3 Remove the instrument and rinse thoroughly under running tap water. Actuate the handle several times during rinsing.
- 4 Place the LIGACLIP Clip Applier in an ultrasonic cleaner with fresh enzymatic detergent for a minimum of 10 minutes. Ensure that the distal end of the instrument is pointed downward. Rinse thoroughly under running tap water while actuating the handle.
- 5 Place the LIGACLIP Clip Applier in an automated washer or washer/disinfector. The instrument should be run through the "instrument" cycle, or a cycle that includes a lubrication step. If lubrication is not an option, the device may be lubricated manually prior to each use.
- 6 The LIGACLIP Clip Applier can be put through an automated thermal disinfection cycle, provided that temperatures remain under 100°C.
- 7 Drying can be accomplished through the automated cycle, provided that temperatures remain under 134°C.

#### **Steam Sterilization**

Following the cleaning and disinfection steps above, the LIGACLIP Clip Applier must be steam sterilized using one of the cycle parameters listed below. Post-sterilization drying time at 134°C or less for a maximum of 30 minutes is allowable. Handling of the LIGACLIP Clip Applier should follow hospital protocol throughout the cleaning and sterilization processes.

**Steam**

The following steam sterilization cycle parameters are approved for use.

Sterilizer Type	Method	Cycle Time (at temperature)	Temperature Set Points	Preconditioning Pulses	Drying Time
Prevacuum	Wrapped	3–18 minutes	270°F / 273°F (132°C / 134°C)	3	30 minutes
Prevacuum	Unwrapped	3–5 minutes	270°F / 273°F (132°C / 134°C)	3	0 - 30 minutes
Gravity	Wrapped	15 minutes	270°F / 273°F (132°C / 134°C)	Not applicable	30 minutes
Gravity	Wrapped	50 minutes	250°F (121°C)	Not applicable	30 minutes
Gravity	Unwrapped	3-10 minutes	270°F / 273°F (132°C / 134°C)	Not applicable	0 - 30 minutes

Note:

- The above table includes the minimum temperature and time validated to assure sterility.
- Based on steam autoclave tolerances, the actual autoclave temperature can exceed the set point temperature by a maximum of +5 °F (+3 °C).
- Health Authorities in some regulated regions do not accept unwrapped sterilization methods. Please review the appropriate guidelines, standards and national Health Authorities' guidelines when determining acceptable steam sterilization process parameters for use in each respective country.**

**Warnings and Precautions**

- Minimally invasive procedures should be performed only by persons having adequate training and familiarity with minimally invasive techniques. Consult medical literature relative to techniques, complications and hazards prior to performance of any minimally invasive procedure.
- Minimally invasive instruments may vary in diameter from manufacturer to manufacturer. When minimally invasive instruments and accessories from different manufacturers are employed together in a procedure, verify compatibility prior to initiation of the procedure.
- A thorough understanding of the principles and techniques involved in laser, electrosurgical, and ultrasonic procedures is essential to avoid shock and burn hazards to both patient and medical personnel and damage to the device or other medical instruments. Ensure that electrical insulation or grounding is not compromised. **Do not** immerse electrosurgical instruments in liquid unless the instruments are designed and labeled to be immersed.
- Ensure that the clip is the correct size for the vessel or tubular structure being ligated.
- Upon each clip application, fully squeeze the trigger of the instrument until it comes to a stop. Failure to fully squeeze the trigger can result in improper formation of clips, and therefore, unsatisfactory ligation of tissue.
- Ensure that each clip has been securely and completely positioned around the tissue being ligated.
- Do not close the jaws of the instrument over other surgical instruments.
- Appliers in disrepair should not be used. The use of such appliers could result in inadequate function of the clip.
- The condition of the LIGACLIP ligating applier, the closure force exerted by the surgeon, the thickness of the tissue to be ligated, and the nature of the clip itself are all critical to the performance of any ligating clip.
- As with any ligating method, inspect each ligation site following placement of the clip to ensure that the clip has been satisfactorily placed.
- Instruments or devices which come into contact with bodily fluids may require special disposal handling to prevent biological contamination.

**How Supplied**

The LIGACLIP Endoscopic Clip Applier is supplied non-sterile.

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ETHICON ENDO-SURGERY, LLC ©EES, LLC 2011
Ethicon Endo-Surgery (Europe) GmbH