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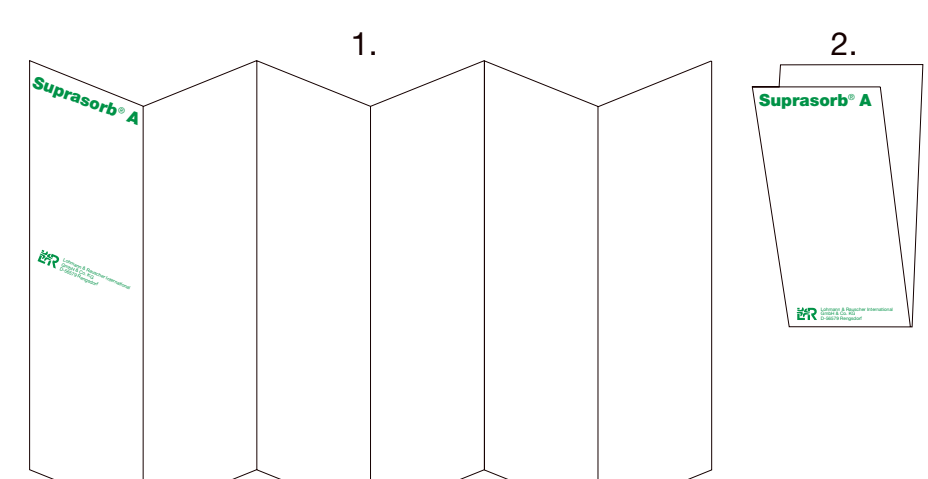
the treatment of heavily exuding and deep wounds which may also be infected. The calcium alginate fibres react with wound exudate or blood and form a gel which creates a moist and humid environment. The calcium alginate Wound Dressing and Rope are sterilized by gamma irradiation and must not be re-sterilized. The products are sterile unless the package is opened or damaged.
Product composition:
 Calcium alginate fibres
Indications:
 Suprasorb® A Calcium Alginate Wound Dressing and Rope are for single use only and can be used for the treatment of the following wounds:
 • pressure sores
 • arterial ulcers
 • venous lower leg ulcers
 • diabetic ulcers
 • skin donor sites
 if they are
 • heavily exuding and
 • superficial wounds or
 • deep wounds.
Contraindications:
 • not suitable for dry wound conditions
 • not suitable for 3rd degree burns
Adverse reactions:
 None known to date.
Instructions for Use:
 a) Place the dressing on the moist wound. If necessary, fold the dressing to match the wound size. In case of slightly exuding wounds, moisten to adjust with sterile saline solution.
 b) Cut the rope gently into cavity wounds.
 c) Secure Suprasorb® A Calcium Alginate Wound Dressing with suitable retention (e.g. Suprasorb® H Hydrocolloid Dressing).
3. Dressing change:
 Change the dressing as directed by the clinician in charge or when strike through occurs, or at least after 5-7 days.
Precautions:
 • The product may not be used in the presence of known allergens to one or several of its components.
Presentation:

Product Code	Size	Pieces
20440	5 x 5 cm	10
20441	10 x 10 cm	10
20442	10 x 20 cm	10
20445	30 cm / 2g	5

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