Autosuture™  
**ENDO BABCOCK™**  
SINGLE USE INSTRUMENT  
10000-29698  
**ENGLISH**

**BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.**

**IMPORTANT!**

This device is designed to be used as a single use product. It is a reference to technical documents.

This device was not designed, tested, manufactured or inspected under any conditions of reuse. Reuse of this device may lead to failure and subsequent patient injury. Reinspection and/or reevaluation of this device may create the risk of contamination and patient infection. Do not reuse, reinspect or reevaluate this device.

**DESCRIPTION**  
The Autosuture™ ENDO BABCOCK™ instrument has a 10 mm diameter shaft, 31 cm in length, and is designed for introduction and use through all approximately size Autosuture™ trocar sleeves, or larger size trocar sleeves with the use of a converter. The jaws measure approximately 44 mm in length, with a maximum opening of 34 mm.

**INDICATIONS**  
The Autosuture™ ENDO BABCOCK™ instrument has applications in a variety of general, thoracic, gynecologic, urologic and endoscopic procedures for temporary stapling/tying of tissue and small blood vessels.

**CONTRAINDICATIONS**  
The Autosuture™ ENDO BABCOCK™ instrument is not intended for use except as indicated. It is also not intended for use where Endoscopic techniques are contraindicated.

**WARNINGS AND PRECAUTIONS**  
1. Endoscopic procedures should be performed only by physicians having adequate training and familiarity with endoscopic techniques. Consult the medical literature for complications, hazards and techniques, prior to performing endoscopic procedures.

2. A thorough understanding of the principles and techniques involved in laser and endoscopic procedures is necessary to avoid shock and heat related to both patient and operator. Verify compatibility of endosumation, and ensure that electrical isolation or grounding is not compromised.

3. These devices are powered STEERLE and are intended for use during a SINGLE endoscopic procedure only. DISCARD AFTER USE. DO NOT RESTERILIZE.

4. Excessive clamping pressure may result in damage to tissue or structure.

**CAUTION: Excessive clamping pressure may result in damage to tissue or structure**

1. To release the tissue or structure from the jaws, simply pull the trigger to the handle.

**STORE AT ROOM TEMPERATURE. AVOID PROLONGED EXPOSURE TO ELEVATED TEMPERATURES.**

**DO NOT EXPOSE TO TEMPERATURES ABOVE 130° F. (54° C).**

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