Instrukzioni per l'uso

I

ACCIAIO INOSSIDABILE SUTURA

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DESCRIZIONE

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STAINLESS STEEL WIRE

STERILE NON ABSORBABLE SUTURE

Full details of the product range are contained in the catalogue. Stainless steel sutures comply with the requirements of the European Pharmacopoeia for Sterile Non-Absorbable strands and the United States Pharmacopoeia for Non Absorbable Surgical Sutures, except for an occasional slight deviation in diameter in some gauge sizes.

DESCRIPTION

Stainless steel suture is a monofilament or multifilament non absorbable sterile surgical suture composed of stainless steel. Stainless steel is available in a range of gauge sizes and lengths, non-needed or attached to stainless steel needles of various types and sizes. Stainless steel is also available in presentations containing a range of components in a variety of materials to allow use as supporting sutures or tendon sutures. It is also available coated with polyethylene.

Full details of the product range are contained in the catalogue. Stainless steel suture complies with the requirements of the European Pharmacopoeia for Sterile Non-Absorbable strands and the United States Pharmacopoeia for Non Absorbable Surgical Sutures, except for an occasional slight deviation in diameter in some gauge sizes.

INDICATIONS

Stainless steel sutures are for use in abdominal wound closure, hernia repair, sternal closure and orthopaedic procedures including cerclage and tendon repair.

Grasping in the point area could impair the penetration performance and cause fracture of the needle. Grasping at the butt or attachment end could cause bending or breakage. Reshaping needles may cause them to lose strength and be less resistant to bending and breaking. Users should exercise caution when handling surgical needles to avoid inadvertent needle stick injury. Discard used needles in "Sharps" containers.

PERFORMANCE

Stainless steel suture elicits a minimal, initial inflammatory reaction in tissue and is not absorbed.

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STATUS 9/02 **RMC 8751205**

ADVERSE REACTIONS

Adverse reactions associated with the use of this device include allergic response in some patients with known sensitivities to stainless steel or constituent metals such as chromium and nickel, minimal initial inflammatory tissue reaction and transient local irritation at the wound site. Like all foreign bodies stainless steel may potentiate an existing infection.

STERILITY

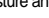
Stainless steel sutures are sterilized by irradiation. Do not resterilize! Do not use if package is opened or damaged! Discard opened, unused sutures!

STORAGE

Recommended storage conditions: Below 25° C, away from moisture and direct heat. Do not use after expiry date!

SYMBOLS USED ON LABELLING

 = Do not reuse

 = Use until Year & Month

 = Sterile unless the package is damaged or opened.

Method of Sterilization – Irradiation

CE 0086 = CE-mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

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