

# STEP-BY-STEP GUIDE




## ReliaTack™ Articulating Reloadable Fixation Device for Standard and Deep Purchase Tacks

### LOADING

Before loading, the ReliaTack™ device must be in reload mode, with the red toggle button exposed.




1  
Ensure that the device is in the straight, unarticulated position.

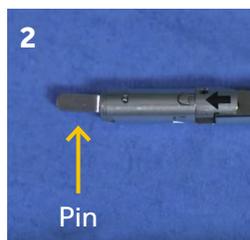

2  
Pin must be in the vertical position. If not vertical, push the red fire button, cycle the device until the pin is vertical, and push the green load button once vertical.




3  
Remove the reload from the blister by grabbing the distal end of the reload and lifting up. Important: Do not grab the shipping wedge.

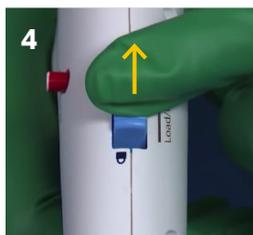


4  
Pull and hold the blue lock switch button in the back position.



5  
Hold the blue lock switch button and insert the pin located at the distal end of the shaft into the reload.

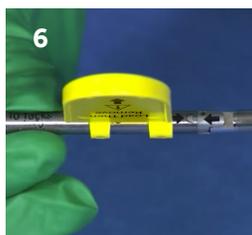

6  
Ensure that the arrow on the shaft is aligned with the arrow on the reload. Push the reload onto the distal end of the shaft until the two arrows meet.

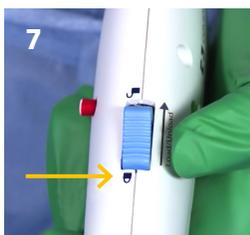


7  
Release the blue lock switch button, allowing it to return to the forward position.




8  
Gently pull on the reload to confirm that the device is loaded.

### UNLOADING

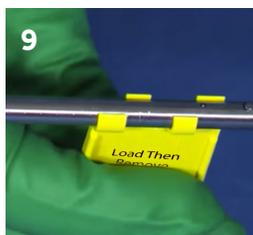


9  
Remove the shipping wedge once the reload is properly locked and loaded.

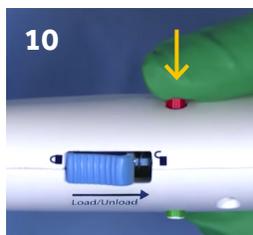


10  
Push the red fire button and hand the device to the surgeon.



1  
Push the green load button to expose the red portion of the toggle button.

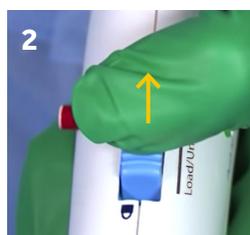

2  
Pull and hold the blue lock switch button in the back position.

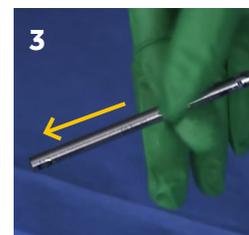

3  
Pull the reload off of the shaft of the device and discard. Important: Do not reuse.